

## Hydrogel Dressings

**Hydrogels:** Primary dressing for wounds with light or no exudate to add or maintain moisture at the wound bed. Helps maintain a moist wound healing environment to assist in autolytic debridement. To protect peri-wound skin from maceration, apply a moisture barrier ointment or barrier film. Available in amorphous, impregnated, and sheet gauze forms.

**Impregnated Gauze:** Contains 20% glycerin and 79% water

**Sheet:** Contains Poly (Ethylene Oxide), Synonym PEO – 4%; De-Ionized Water – 96%

**Amorphous:** Contains 1.5% glycerin and 94.66% water

**Indications:** Provide moist wound healing environment and help promote autolytic debridement.

**Contraindications:** Patients with a known hypersensitivity to the product itself or to its components.

**Change Frequency:** Hydrogel Amorphous Dressings should be changed daily or as directed by a healthcare professional. Other Hydrogel Dressings should be changed as directed by a healthcare professional.

**Recommended Secondary Dressings:** Cover with an appropriate secondary dressing depending on the amount of exudate and frequency of dressing change.



### Satisfaction Guaranteed

If you are not completely satisfied with any McKesson Brands product, you may return it for a full refund or credit.

### Never Compromise on Care<sup>SM</sup>

With McKesson Brands, you'll never have to compromise. We deliver quality products at competitive prices, backed by our 100% satisfaction guarantee. We have decades of experience in the market, and we offer you support and work to help ensure you have the products you need, when you need them – making it even easier for you to provide the best care for your patients.

The product information contained in this document, including the product images and additional product materials, was collected from various supplier sources. All product claims and specifications are those of the product suppliers and have not been independently verified by McKesson Medical-Surgical or its affiliates ("McKesson"). McKesson is not responsible for errors or omissions in the product information.

The properties of a product may change or be inaccurate following the posting or printing of the product information in the document, either in the print or online version. Caution should be exercised when using or purchasing any products from McKesson's online or print documents by closely examining the product packaging and the labeling prior to use.

Due to product changes, information listed in this document is subject to change without notice. This information is placed solely for your convenience in ordering and McKesson disclaims all responsibility for its completeness and accuracy, whether or not the inaccuracy or incompleteness is due to fault or error by McKesson.

## ORDERING INFORMATION

DESCRIPTION	MFR #	QTY	HCPCS	EXUDATE LEVEL*
Hydrogel Amorphous Wound Dressing, Non-Sterile, 1 fl oz.	1721	1/ea, 30/cs	A6248	●○○
Hydrogel Amorphous Wound Dressing, Non-Sterile, 3 fl oz.	1722	1/ea, 12/cs	A6248	●○○
Hydrogel Impregnated Gauze Dressing, Sterile, 2" x 2"	61-53022	1/ea, 10/bx	A6231	●○○
Hydrogel Impregnated Gauze Dressing, Sterile, 4" x 4"	61-53044	1/ea, 10/bx	A6231	●○○
Hydrogel Sheet Dressing, Sterile, 2" x 2"	61-82022	1/ea, 10/bx	A6242	●○○
Hydrogel Sheet Dressing, Sterile, 4" x 4"	61-82044	1/ea, 10/bx	A6242	●○○

EXUDATE LEVEL KEY: ●○○ = Light ●●○ = Moderate ●●● = Heavy

\*Exudate level indicates highest level of exudate that the dressing can accommodate